FT 100

DE	Kontaktloses Thermometer Gebrauchsanweisung	2-22
GB	Non-contact thermometer Instructions for use	23-43
FR	Thermomètre sans contact Mode d'emploi	44-65
ES	Termómetro sin contacto Instrucciones de uso	66-86
IT	Termometro senza contatto Istruzioni per l'uso	87-107
TR	Temassız Termometre Kullanım Kılavuzu	108-127
RUS	Бесконтактный термометр Инструкция по применению	128-150
PL	Termometr bezdotykowy Instrukcja obsługi	151-171

ELECTROMAGNETIC COMPATIBILITY INFORMATION	172-175
--	----------------

Inhalt

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Germany

Contents

1. Signs and symbols	24
2. Intended use.....	25
3. Warnings and safety notes	26
3.1 Risks to the user	26
3.2 Risks to the device	27
3.3 Notes on handling batteries	27
4. Information about this thermometer.....	28
5. Device description.....	30
6. Initial use	31
7. Switching on and setting the thermometer	32
8. Forehead measurement.....	36
9. Measuring object temperature/room temperature ..	38
10. Error messages	39
11. Batteries	40
12. Cleaning the device.....	41
13. Storing the device	41
14. Disposing of the device.....	41
15. Technical data	42
16. Guidelines	43

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage and air. Please read these instructions for use carefully and observe the information they contain.

With kind regards,
Your Beurer team

Included in delivery

Check that the delivery has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- Clinical thermometer
- 2x 1.5 V AAA batteries (LR03)
- Storage bag
- Instructions for use

1. Signs and symbols

The following symbols appear in these instructions for use and on the device:

	WARNING Warning notice indicating a risk of injury or damage to health.
	IMPORTANT Safety note indicating possible damage to the device/accessory.
	Note Note on important information.
	Observe the instructions for use
	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices.

	Manufacturer
Storage 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity
IP 22	Device protected against foreign objects ≥ 12.5 mm and against water dripping at an angle
	Serial number
	Important information/tip
	Cross-reference with another section
	Certification symbol for products that are exported to the Russian Federation and CIS countries.

2. Intended use

This thermometer is intended for non-contact measurement of forehead, object and room temperature. The thermometer is suitable for domestic use. It allows you to quickly and easily measure body temperature. Values are saved automatically.

3. Warnings and safety notes

3.1 Risks to the user

- Only use the device once you have read and understood these instructions for use.
- Retain these instructions for use. The instructions for use must be accessible to all users. All instructions must be followed.
- The thermometer needs to be in the room in which the measurement is taken for at least 30 minutes before use.
- After physical activity, a bath or spending time outdoors, wait approx. 30 minutes before you take a measurement on your forehead.
- The FT 100 thermometer is only designed for the measuring area on the human body stated in the instructions for use.
- Do not self-diagnose or self-medicate on the basis of the measurements without consulting your doctor. In particular, do not start taking any new medication or change the type and/or dosage of any existing medication without prior approval.
- The device is only intended for the purpose stated in these instructions for use.
- The manufacturer is not liable for damage resulting from improper or careless use.
- Children must not be allowed to use the device. Medical devices are not toys.
- Check before each use that the lens is intact. If it is damaged, please contact your retailer or the service address.
- Do not touch the lens with your fingers.
- The measurements taken by you are for your information only – they are no substitute for a medical examination. Consult a doctor before you undertake any medical treatment.
- Clean the thermometer after each use (▷ 12. Cleaning the device).
- Should you have any questions about using the device, please contact either your retailer or Customer Services.

3.2 Risks to the device

- Do not drop the device or subject it to any mechanical impacts.
- Do not expose the device to direct sunlight, extreme temperatures or high humidity.
- Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
- Do not open or repair the device yourself. Have the device repaired by authorised service centres only, otherwise its warranty is invalidated.
- Portable and mobile HF communication systems may interfere with this device. More details can be requested from the stated Customer Services address or found at the end of the instructions for use.

3.3 Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected area with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

4. Information about this thermometer

The temperature measurement varies depending on the part of the body where the measurement is taken. In a healthy person, the variance can be between 0.2°C and 1°C (0.4°F and 1.8°F) in different parts of the body.

Normal temperature range with various thermometers:

	Measurements	Thermometer used
Forehead temperature	35.8°C to 37.6°C (96.4°F to 99.7°F)	Forehead thermometer
Ear temperature	36.0°C to 37.8°C (96.8°F to 100.0°F)	Ear thermometer
Oral temperature	36.0°C to 37.4°C (96.8°F to 99.3°F)	Conventional thermometer
Rectal temperature	36.3°C to 37.8°C (97.3°F to 100.0°F)	Conventional thermometer

Beurer tip

- Temperatures measured with different thermometers should never be compared with one another.
- Tell your doctor what type of thermometer you used to take your temperature and on what part of the body. Also bear this in mind if you are diagnosing yourself.

Holding the thermometer in your hand for too long before taking a measurement can cause the device to warm up. This means the measurement could be incorrect.

Influences on body temperature

- A person's individual metabolism
- Age
Body temperature is higher in babies and toddlers than in adults. Greater temperature fluctuations occur faster and more often in children. Normal body temperature decreases with age.
- Clothing
- Outside temperature
- Time of day
Body temperature is lower in the morning and increases throughout the day towards evening.
- Activities
- Physical and, to a lesser extent, mental activities increase body temperature.


Beurer tip

Taking the body temperature provides a current measurement of a person's temperature. If you are uncertain about interpreting the results or if the values are abnormal (e.g. fever), please consult your doctor. This also applies in the case of slight temperature changes if there are other symptoms of illness such as agitation, heavy sweating, flushed skin, fast pulse rate, tendency to collapse, etc.

5. Device description


- 1 Display
- 2 **MODE/MEM** button
- 3 Fever/distance indicator
- 4 **LIGHT/SET** button
- 5 Measuring sensor
- 6 Distance sensor
- 7 **ⓘ/SCAN** button
- 8 Battery compartment lid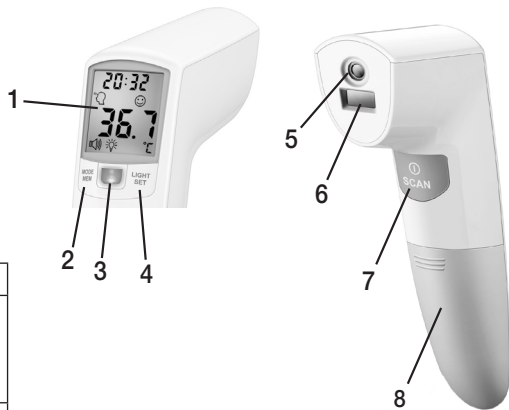

Buttons	Functions
MODE/MEM	MODE Sets the measurement mode. MEM Displays the saved measurements.
LIGHT/SET	LIGHT Manual activation of illuminated display. SET Sets the basic functions.
ⓘ SCAN	Turns the device on and off. Starts the temperature measurement.

Display description

- 1 Year/date/time display
- 2 Room temperature mode
- 3 Forehead temperature mode
- 4 Object temperature mode
- 5 Temperature/memory space number display
- 6 Acoustic signal symbol
- 7 Illuminated display symbol
- 8 Year
- 9 Memory function
- 10 Date
- 11 Measurement $\geq 38.0^{\circ}\text{C}$ ($\geq 100.4^{\circ}\text{F}$) "fever"
- 12 Measurement $< 37,5^{\circ}\text{C}$ ($< 99,5^{\circ}\text{F}$) "no fever"
- 13 Battery status indicator
- 14 Temperature measurement unit (Celsius/Fahrenheit)

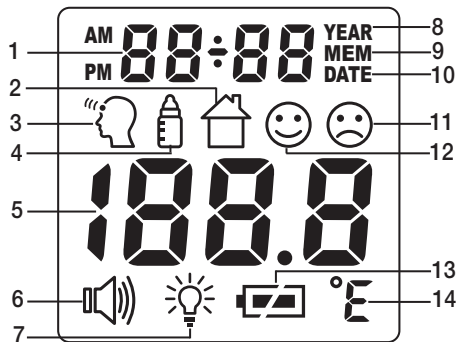6. Initial use

If present, remove the battery insulating strip on the battery compartment lid and remove the protective film from the battery, then insert the battery the correct way round.

- Push open the battery compartment lid.
- Insert the two batteries included in delivery so that they are the correct way round.
- Push the battery compartment lid back onto the thermometer until you can feel and hear it click firmly into place.


- The thermometer now automatically goes to the settings menu (▷ 7. Switching on and setting the thermometer).

7. Switching on and setting the thermometer

To switch on the thermometer, briefly press the **⊙/SCAN** button.

After a brief self-test and two short beeps the thermometer is ready for forehead measurement. The device always starts up in forehead temperature mode .

Setting the basic functions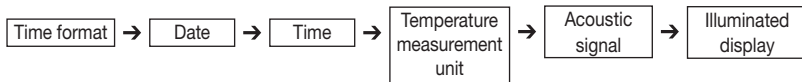

This menu allows you to set the following functions individually, one after another.

Time format

- Insert the batteries or press and hold the **LIGHT/SET** button for 4 seconds when the thermometer is switched on. The time format flashes on the display (Fig. 1).
- Use the **MODE/MEM** button to set your preferred time format, and confirm with the **LIGHT/SET** button.

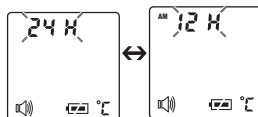
Fig. 1

The year flashes on the display (Fig. 2).

- Use the **MODE/MEM** button to set the year, and confirm with the **LIGHT/SET** button.

The day/month flashes on the display (Fig. 3).

- Use the **MODE/MEM** button to set the day and month, and confirm with the **LIGHT/SET** button.

- ① If the hour format is set as 12 h, the day/month display sequence is reversed. Be sure to set the date and time correctly. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

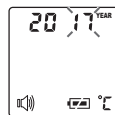
Fig. 2

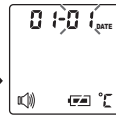
Fig. 3

The hour flashes on the display (Fig. 4).

- Use the **MODE/MEM** button to set the hour, and confirm with the **LIGHT/SET** button.

The minute flashes on the display (Fig. 5).

- Use the **MODE/MEM** button to set the minute, and confirm with the **LIGHT/SET** button.

- ① In the 12 h format, the time is displayed with AM/PM.

Fig. 4

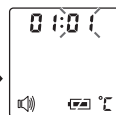
Fig. 5

The temperature measurement unit flashes on the display (Fig. 6).

You can set the device to display the temperature in degrees Celsius (°C) or degrees Fahrenheit (°F).

- To display the temperature in Celsius, select °C using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To display the temperature in Fahrenheit, select °F using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

Fig. 6

The acoustic signal symbol flashes on the display (Fig. 7).

You can activate/deactivate the acoustic signals (device activation, measurement in progress, measurement completion).

- To activate the acoustic signals, use the **MODE/MEM** button to select 00, and confirm with the **LIGHT/SET** button.
- To deactivate the acoustic signals, use the **MODE/MEM** button to select 0FF, and confirm with the **LIGHT/SET** button.

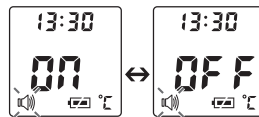
Fig. 7

The illuminated display symbol flashes on the display (Fig. 8).

You can activate or deactivate the automatic illuminated display (appears following forehead temperature measurement).

- To activate the automatic illuminated display, use the **MODE/MEM** button to select **ON**, and confirm with the **LIGHT/SET** button.
- To deactivate the automatic illuminated display, use the **MODE/MEM** button to select **OFF**, and confirm with the **LIGHT/SET** button.

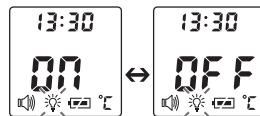
Fig. 8

You can also activate the illuminated display manually by briefly pressing the **LIGHT/SET** button. The display lights up for 5 seconds.

- ⓘ The automatic and manual illuminated display cannot be used while measurement is in progress.

8. Forehead measurement

Beurer tip

Remember:

- Physical activity, increased perspiration on the forehead, taking vasoconstrictive medication and skin irritations can distort the measurement.
- The forehead, or the temples, must be free from perspiration and cosmetics.
- We recommend that you take three measurements on the forehead. If the three measurements are different, use the highest value.

- To switch on the thermometer, briefly press the **⏻/SCAN** button. After a short self-test, the device is ready for forehead temperature measurement. The device is in forehead temperature mode. You can tell this from the  symbol on the display.

Fig. 9

- Hold the thermometer 4 to 6 cm in front of the forehead measuring point (Fig. 10). Press the **⏻/SCAN** button. The display shows the required measuring distance of 4-6 cm.

Fig. 10

The measuring distance is 4-6 cm	The measuring distance is NOT 4-6 cm
Once the thermometer has detected the correct measuring distance of 4 to 6 cm, the measurement starts automatically. The fever/distance indicator now lights up continuously in blue. After a few seconds, the measurement appears on the display (Fig. 12). The end of the measurement is signalled by a long beep.	If the measuring distance is not 4 to 6 cm, the fever/distance indicator flashes blue (the device also beeps if the acoustic signal is switched on) and 4-6 appears on the display (Fig. 11). If the correct measuring distance is not found after 20 seconds, the measurement ends automatically.

❗ If the measurement is less than 37.5°C (99.5°F), if the fever/distance indicator lights up in green and if 😊 appears on the display to indicate no fever, this indicates that the body temperature is in the normal range.

If the measurement is between 37.5°C and 37.9°C, the fever/distance indicator lights up yellow. This indicates that the body temperature is in an increased range (Fig. 13).

If the measurement is more than 38°C (100.4°F), if the fever/distance indicator lights up in red and if ☹ appears on the display (Fig. 14), this indicates that the body temperature is above the normal range.

The measurement is automatically saved with the date/time and the “fever” 😞 / “no fever” 😊 classification.

The device is ready to take another measurement when two short beeps sound and the forehead symbol 🧠 stops flashing.

Fig. 11

Fig. 12

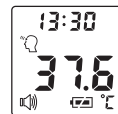
Fig. 13

Fig. 14

Displaying stored measurements

The device only stores measurements in forehead temperature mode . The device automatically stores the values from the last 60 measurements.

When 60 memory spaces are exceeded, the oldest value is deleted.

- The memory can be called up as follows:
- With the thermometer switched on, press and hold the **MODE/MEM** button for 5 seconds. The most recent measurement is displayed.
- With each subsequent pressing of the **MODE/MEM** button, first the memory space number is displayed, followed by the measurement when the button is released.
- In the upper line, the time and date are displayed alternately.

9. Measuring object temperature/room temperature

Object temperature mode

If you want to measure an object temperature with this thermometer, switch to object temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button. The device switches to the object temperature mode  (Fig. 15).
- Hold the thermometer 2 to 3 cm from the measuring point. Briefly press the **SCAN** button and read the temperature on the display (Fig. 16).

Measurements taken in object temperature mode are not stored.

Fig. 15

Fig. 16

If you want to use the thermometer to measure the room temperature you need to switch to room temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button twice.

The device switches to room temperature mode .

- The room temperature is immediately displayed (Fig. 17).

Measurements taken in room temperature mode are not stored.

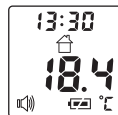
Fig. 17

10. Error messages

Error message	Problem	Solution
<i>Er 1</i>	Measurement during self-test, device not yet ready for measurement.	Wait until the forehead symbol stops flashing.
<i>Er 3</i>	Room temperature below 10°C or above 40°C (<50°F, >104°F).	Room temperature must be between 10°C and 40°C (50°F - 104°F).
<i>Hi</i>	(1) Forehead temperature mode: The temperature recorded is higher than 42.2°C (108°F). (2) Object temperature mode: The temperature recorded is higher than 80°C (176°F).	Operate the thermometer only within the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
<i>Lo</i>	(1) Forehead temperature mode: The temperature recorded is lower than 34°C (93.2°F). (2) Object temperature mode: The temperature recorded is lower than -22°C (-7.6°F).	Operate the thermometer only within the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.

Error message	Problem	Solution
	The batteries are empty.	Replace the batteries.

11. Batteries

Changing the batteries

The device requires two AAA (LR03) batteries.

- Open the battery compartment.
To do this, push the battery compartment down.
- Take the used batteries out of the battery compartment.
- Insert new batteries.
Make sure that the batteries are inserted the correct way round.
- Close the battery compartment.

Disposing of batteries

- Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead
Cd = Battery contains cadmium
Hg = Battery contains mercury

12. Cleaning the device

-  • The measuring sensor is the most sensitive part of the thermometer. Be very careful with the measuring sensor when cleaning the device.
 - Do not use any harsh cleaning products.
 - Always observe all safety notes for users and the device.
- ▷ 3. Warnings and safety notes
-

Clean the measuring sensor after each use. Use a soft cloth or cotton bud that can be moistened with disinfectant or 70% alcohol.

To clean the entire device, please use a soft cloth slightly moistened with a mild soapy solution. Under no circumstances may liquid enter the device.

Do not use the device again until it is completely dry.

13. Storing the device

The device must not be stored or used at an excessively high or low temperature or humidity (▷ 15. Technical data), in sunlight, in association with an electrical current or in dusty locations. Measurement inaccuracies can otherwise occur.

If prolonged storage is planned, you should remove the batteries.

14. Disposing of the device

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

15. Technical data

If the device is not used according to the instructions specified, perfect functionality cannot be guaranteed!
The accuracy of this thermometer has been carefully checked and developed with regard to a long useful life.
If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means.
Precise instructions for checking accuracy may be requested from the service address.
We reserve the right to make technical changes to improve and develop the product.

Type	FT 100
Measurement method	Non-contact infrared measurement
Basic functions	Forehead temperature measurement Object temperature measurement Room temperature measurement
Measurement units	Celsius (°C) or Fahrenheit (°F)
Operating conditions	10°C to 40°C (50°F to 104°F) at a relative humidity of <95%
Storage and transport conditions	-20°C to 50°C (-4°F to 122°F) at a relative humidity of <95%
Measuring distance	4 to 6 cm from the measuring point
Measurement range and accuracy of forehead temperature measurement	Forehead temperature measurement 34°C to 42.2°C (93.2°F to 108°F) Measurement accuracy 34°C to 42.2°C: $\pm 0.2^{\circ}\text{C}$ (93.2°F to 108°F: $\pm 0.4^{\circ}\text{F}$)
Clinical repeat precision	0.23°C (0.41°F)
Measurement range and accuracy Object temperature measurement	Object temperature measurement -22°C to 80°C (-7.6°F to 176°F) Measurement accuracy $\pm 4\%$ or $\pm 2^{\circ}\text{C}$ ($\pm 4^{\circ}\text{F}$)

Memory function Body temperature measurement	Automatically stores the last 60 measurements.
Acoustic signal	With device activation, during measurement, when measurement is complete (can be set).
Display	LCD display
Energy saving functions	Device automatically switches off after 1 minute.
Dimensions Width x depth x height	Approx. 40.9 mm x 68.2 mm x 199.0 mm
Weight	82 g (without batteries)
2 x AAA (LR03) batteries	Batteries last for approx. 2000 measurements Extra activated functions such as acoustic signal or illuminated display reduce the battery life.

The serial number is located on the device or in the battery compartment.

16. Guidelines

This device complies with EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktegesetz), the ASTM (American Society for Testing and Materials) E 1965 - 98 and the European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility.

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Electromagnetic Compatibility Information

Table 1
For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration – electromagnetic emissions		
The FT 100 is intended for use in the electromagnetic environment specified below. The customer or the user of the FT 100 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The FT 100 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The FT 100 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Table 2**For all ME EQUIPMENT and ME SYSTEMS**

Guidance and manufacturer's declaration – electromagnetic immunity			
The FT 100 is intended for use in the electromagnetic environment specified below. The customer or the user of the FT 100 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	6 kV contact 8 kV air	6 kV contact 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	2 kV for power supply lines 1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	1 kV line(s) to line(s) 2 kV line(s) to earth	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5 % UT (> 95 % dip in UT) for 0.5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles < 5 % UT (> 95 % dip in UT) for 5 sec	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the FT 100 requires continued operation during power mains interruptions, it is recommended that the FT 100 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Table 3
For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration – electromagnetic immunity

The FT 100 is intended for use in the electromagnetic environment specified below.
 The customer or the user of the FT 100 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 3 V/m 80 MHz to 2.5 GHz	3 Vrms 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the FT 100, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3 \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p>			
<p>NOTE 2 These guidelines may not apply in all situations.</p>			
<p>Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>^a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the FT 100 is used exceeds the applicable RF compliance level above, the FT 100 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the FT 100.</p> <p>^b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.</p>			

Table 4

For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the FT100

The FT 100 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the FT 100 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the FT 100 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany • www.beurer.com
www.beurer-gesundheitsratgeber.com • www.beurer-healthguide.com

758.627 – 0317